

# Liebig-Dispenser

## Evaporator for Formic Acid



Period of best use

1	2	3	4	5	6	7	8	9	10	11	12
---	---	---	---	---	---	---	---	---	----	----	----

- Well suited for standard hives
- With adaptable paper wick for easy dosage
- Easy to fill and to check with imprinted measuring scale.

### Application

- First Application after honey harvest mid-end of June
- Feed the colony after first treatment
- Second treatment in September for 6 – 14 days

### Dosage

- Use 50 – 200 ml of FORMIC Acid per treatment
- Adjust the paper wick respecting hive system and temperature.



#### 1. Adjust

Adjust size of paper wick following the instructions for hive system and temperature.



#### 2. Fill

Remove dropper and fill bottle with the required amount. Insert dropper and close with cap for transport to the hive.



#### 3. Place

Place bottle on Support with adjusted paper wick. Formic acid will soak the paper and evaporate.